

# What's Tested



Chlamydia, Gonorrhea, Trichomoniasis, Mycoplasma genitalium, Syphilis, HIV Type 1, HIV Type 2, Hepatitis B, and Hepatitis C

These are common, treatable, sexually transmitted diseases (STDs) that are easily spread by having vaginal, anal, or oral sex with someone who is infected with the disease.

Note: This test detects the genetic material of the bacteria that cause these STDs in your urine. If you engage in sexual acts that involve sites on your body other than your genitals, such as your mouth or rectum, then this single-site test may be insufficient.

## Need Help?

(877) 760-2333




Feeling nervous? We are here to help.  
Call or text us, and we'll personally walk you through your test.

## Collect Urine Sample

### 1. Prepare

Fill out the transport tube with your full name, today's date, and your date of birth.

Catch your urine in the provided collection cup.

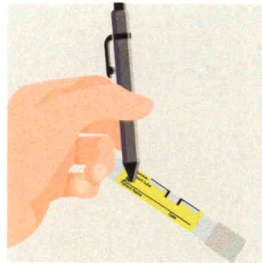

### 2. Transfer urine

Unscrew the cap of the transport tube to prepare it for your sample.

Use the pipette to transfer urine from the collection cup into the transport tube.




### 3. Check level then seal tightly

Keep adding urine until you have reached the minimum fill line.

Do not exceed the maximum fill line.

When you have enough urine, screw the cap on tightly.

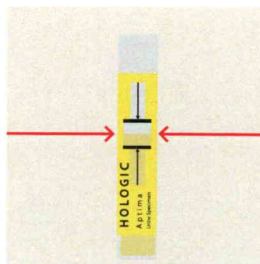

## Return

### Mail back immediately

Place the transport tube into the biohazard bag.

Place the biohazard bag into the prepaid mailer.





**REGISTER  
YOUR KIT  
FIRST**

To register, scan the QR code or visit:  
[tests.mojoon.com.com/register](https://tests.mojoon.com.com/register)



We recommend drinking a glass of water to hydrate before starting.



# Urine Collection

Supplies and Instructions

